




Thank you for purchasing FUJILUX  
Massage Chair.  
This product has a variety of massage  
functions, which can be used to relieve  
fatigue and improve body function.

## MASSAGE CHAIR

instruction manual

Jp-3000

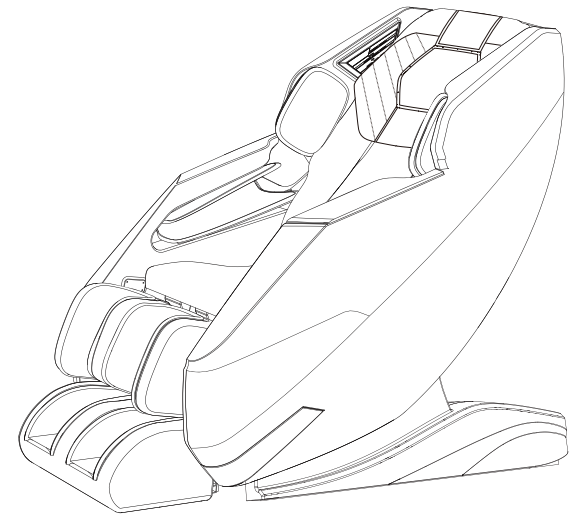



- Please read this manual before use, so as to use the machine correctly.
- Keep this instruction manual in a place convenient for the user to search for, in case of emergency use.
- This machine is only for home use. If it is used for non house hold purposes (such as commercial use), please consult the sales shop or dealer when purchasing this product





# Safety precautions

Please read the safety precautions carefully before use, so as to use the machine correctly. To ensure that you use the machine correctly and safely, and avoid personal injury or property loss to you and others. Please fully understand the following contents (diagrams and marks) and operate strictly.

## Types of illustrations






 Warning!	Wrong operation may cause death or serious injury.
 Be careful!	Incorrect operation may cause personal injury or property damage.

## Types and contents of marks

	 Denotes prohibited behavior. (The left figure shows that disassembly is prohibited)
	 The content of an instruction that represents a coercive act. (The left figure shows pulling the power plug out of the socket)

After reading, please keep this instruction manual in a place convenient for the user to find, in case of emergency.


## Points for attention at the place of use

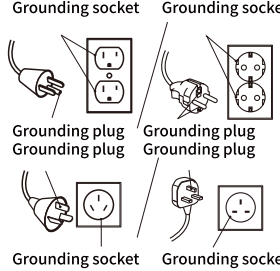 Be careful!	
 Be careful!	<ul style="list-style-type: none"> <li>Do not use it in the bathroom or sauna and other places with heavy humidity. → Otherwise, electric shock or fault may be caused.</li> </ul>
 force	<ul style="list-style-type: none"> <li>Place the machine on a flat surface for use. → Otherwise, the massage chair may topple and cause accidents.</li> </ul>
 Circuit grounding	<ul style="list-style-type: none"> <li>The machine must be grounded. If the machine fails or is damaged, the risk of electric shock can be reduced. A grounding plug is installed in the power cord of the machine, and it can be grounded by inserting the power plug.</li> <li>The power plug must be plugged into a properly installed and grounded socket completely in accordance with local laws and regulations. If you have questions about whether the machine is properly grounded, you can ask a professional to check.</li> <li>The grounding plug of this product is shown in the right figure. If it does not match the socket in your home, you need to connect the adapter sold in the market.</li> </ul> <div style="text-align: right;">  </div>


# Safety precautions

# Safety precautions

## Precautions before use

Warning!	
 <b>Be careful!</b>	<ul style="list-style-type: none"> <li>○ The following persons are not allowed to use this machine. People who are forbidden to receive massage by doctors, such as those suffering from thrombosis (embolism), severe aneurysm, acute venous aneurysm, various skin inflammation and skin infection (including inflammation of subcutaneous tissue).</li> <li>○ Do not use a power supply other than AC 220-240V. → Otherwise, fire or electric shock may be caused.</li> <li>○ Do not use transformers. → Otherwise, failure or electric shock may occur.</li> <li>○ Do not use the machine when the power cord or plug is damaged or the socket is loose. → Otherwise, electric shock, short circuit and fire may be caused.</li> </ul>
	<ul style="list-style-type: none"> <li>● The following persons must consult a doctor before use. <ul style="list-style-type: none"> <li>* cardiac</li> <li>* People with severe peripheral vascular circulation disorder caused by diabetes and other factors, resulting in sensory impairment</li> <li>* People with osteoporosis, spinal fracture and acute pain</li> <li>* People with trauma at the treatment site</li> <li>* People whose body temperature is higher than 38 °C (high fever period), for example, the period of severe acute inflammatory symptoms (tiredness, chills, unstable blood pressure, etc.). When the body is weak.</li> <li>* People in the unstable period of early pregnancy or shortly after delivery</li> <li>* People who are using implantable medical devices that are vulnerable to electromagnetic interference such as cardiac pacemakers</li> <li>* People with abnormal spine and bent spine</li> <li>* People who want to use it on the part that has been treated by a doctor or the part of a disease that is being treated</li> <li>* Patients with malignant tumors</li> <li>* People who must rest</li> <li>* Patients with visceral diseases (gastritis, enteritis, hepatitis) and other acute diseases</li> <li>* People who are being treated by doctors, especially those who feel abnormal</li> <li>* People with rheumatism, gout and other causes of joint deformation</li> <li>* People with dizziness, instability and tinnitus</li> <li>* People with inflammatory reaction due to contusion, muscle strain, etc.</li> <li>* People suffering from tenosynovitis or possibly from tenosynovitis</li> <li>* People considered to have lost the sense of temperature</li> </ul> </li> </ul> <p>Before massage, be sure to confirm whether the fabric at the massage part is cracked or damaged. Please check other parts to ensure that the fabric is not damaged. If you find any damage on the fabric, regardless of the size of the damage, please unplug the power plug and contact the dealer who bought this product.</p> <p>Please never remove the fabric of the massage part or use it when the fabric is damaged, otherwise there will be serious potential safety hazards.</p> <p>→ If the machine is used when the fabric is damaged, it may cause injury or electric shock.</p> <p>The elderly or people with weak bones, even if they do not have specific diseases should consult a doctor before using the massage chair</p> <p>If it has no effect after use, please consult a doctor or an expert.</p> <p>Do not let children and pets use or play with the machine.</p>

## Precautions before use

Be careful!	
 <b>prohibit</b>	<ul style="list-style-type: none"> <li>● Do not use the machine in a way not described in this manual. In addition, please do not use this machine with other treatment equipment, or use it after covering with electric blanket → Otherwise, it may cause adverse effects on the body or injury.</li> <li>● Please do not move the massage chair under manned condition. → Otherwise, the massage chair may fall over and cause accidents or injuries.</li> <li>● Do not sit on the chair with the back flat. → Otherwise, excessive pressure may be brought to the leg components, resulting in failure. In addition, an accident or injury may occur if the massage chair collapses.</li> <li>● Do not use anything other than accessories when using this machine.</li> </ul>
	<ul style="list-style-type: none"> <li>● Before inserting the power plug into the socket, make sure that the main power switch is in the "O" state. → Otherwise, failure or electric shock may occur.</li> <li>● Please fully insert the power plug and connector. → Otherwise, electric shock, short circuit and fire may be caused.</li> <li>● Before use, please follow this instruction manual, turn on all switches in turn, and confirm the normal operation of the machine.</li> <li>● At the beginning, please start the massage with a light massage program. → At the beginning, excessive massage stimulation may cause injury. Especially for the elderly or those with weak bones, please pay attention to massage.</li> <li>● For safety, please confirm the position of the device. If the device is not in the storage position, please turn on the main power switch, and then slowly sit down after confirming that it has been stored.</li> <li>● Do not squat or sit on the calf-rest assembly. → Massage chair tilting may cause accidents, injuries or failures.</li> <li>● If the machine has not been used for a long time, please re read this instruction manual before using it again to confirm that the machine can operate normally.</li> </ul>

# Safety precautions

# Safety precautions

## Precautions before use



### Warning!



prohibit

- Do not wind the power cord on the massage chair or step on the body and calf-rest assembly. In addition, please do not damage, damage, process, over bend, stretch or twist the power cord  
→ Otherwise, the wire may be damaged, causing fire or electric shock.
- Do not spill water or other liquids on the massage chair.  
→ Otherwise, electric shock, short circuit and fire may be caused.
- Do not sit on the massage chair or operate with wet hands when there is water on your body.  
→ Otherwise, electric shock or fault may be caused.
- Do not allow children, people unable to express their own will or people with limited mobility to use the massage chair. In addition, do not let children or pets play on or around the massage chair (behind the chair back, under the leg components, etc.), or squat or sit on the massage chair.  
→ Otherwise, accidents or injuries may be caused
- Do not use the massage chair when more than 2 people are sitting on the chair. Especially when holding a child (baby, etc.).  
→ Otherwise, accidents or injuries may be caused
- Stop using if the fabric is damaged, and pull out the power plug from the socket  
→ Otherwise, accidents, injuries or electric shock may result. For maintenance, please contact the dealer when purchasing this product
- Do not touch the LED directly when the light is on or just off.  
→ Otherwise, electric shock or fault may be caused.



force

- Before starting the massage and when standing up and lowering the back of the chair, make sure there are no people (especially children) and pets around. In addition, please be careful not to clamp your body or objects behind the chair or under the leg components.  
→ Otherwise, accidents or injuries may be caused
- When your body or object is caught, please press the seat back adjustment button to return the seat back to a safe position.
- Do not cover or touch the LED part with cloth, paper, plastic stool and other flammable articles.
- Do not insert metal objects ( hairpin , pins) into the gap of the machine (especially the LED part and power cord).  
→ Otherwise, electric shock, short circuit or fire may be caused.

## Precautions before use



### Be careful!



prohibit

- Do not use the massage chair directly with bare skin.
- Do not use the massage chair for purposes other than massage and chair.  
→ Otherwise, accidents or injuries may be caused
- Do not put your hands in the movable parts of the massage wheel, such as the up and down motion parts, the chair back, etc. In addition, do not wear hard objects such as belts when using.  
→ Otherwise, it may cause injury or failure.
- Do not use on the head, chest, abdomen, joints (elbows, knees, etc.).  
→ Otherwise, it may cause physical discomfort or injury.
- Do not operate the massage chair when nobody is around.  
→ Otherwise, accidents or injuries may be caused
- The calf-rest assembly can only be used for legs.  
→ Otherwise, it may cause injury.
- When massaging the leg, do not put the leg in again forcibly after the leg is pulled out of the calf-rest assembly.  
→ If forced, it may cause failure.
- The armrest assembly can only be used on the wrist.  
→ Otherwise, it may cause injury.
- When massaging the wrist, do not force it into the wrist after the wrist is extruded from the armrest assembly.  
→ Forced insertion may cause failure.
- Do not look directly at the LED of the side panel assembly  
→ Otherwise it may cause injury.





force

- In case of emergency or abnormal operation of the massage chair, please press the [Safety Stop] button on the remote control to stop the massage.
- If you feel severe pain or physical abnormalities during use, please stop using immediately and consult a doctor.
- If you feel that the massage is too stimulating, please reduce the irritation according to this manual. If the stimulation is still too strong after adjustment, please stop using and contact the dealer who bought the product (recorded on the back cover).  
→ If you continue to use the product even though it is too stimulating, you may get injured.
- Please avoid falling asleep during massage  
→ Otherwise it may cause injury.
- The massage time shall not exceed 40 minutes every day. Do not massage the same place for more than 5 minutes.  
→ Continuous use for a long time will cause excessive stimulation, resulting in adverse effects or injuries. Please control the total massage time within 60 minutes every day
- Before use, make sure there are no people (especially children) and pets around.  
→ Otherwise, it may cause injury or failure.
- Do not remove the lower back cushion during massage.





# Safety precautions

# Safety precautions





## Precautions before use

Be careful!	
	
 Unplug the power plug	<ul style="list-style-type: none"> <li>● In case of power failure or possible power failure, please stop using immediately, turn off the power switch and unplug the power plug from the socket. → Otherwise, accidents or injuries may occur when the power supply is restored.</li> <li>● When thunder may occur, please stop using immediately, turn off the power switch, and unplug the power plug from the socket.</li> <li>● When the massage chair does not work or feels abnormal, to prevent accidents, please stop using it immediately, turn off the power switch, unplug the power plug from the socket, and contact the dealer who purchased the product for maintenance. → Otherwise, electric shock, electric leakage or short circuit may be caused, which may cause fire.</li> </ul>

## Precautions after use and when not in use

Be careful!	
	
 prohibit	<ul style="list-style-type: none"> <li>● Do not use the power cord for equipment other than this machine.</li> <li>● After using this machine, make sure the seat back is completely upright.</li> <li>● Do not put articles on the massage chair</li> <li>● Do not replace the LED lamp.</li> </ul>
 force	<ul style="list-style-type: none"> <li>● After use, be sure to turn off the power switch and unplug the power plug from the socket. → Otherwise, accidents or injuries may be caused by children or pets.</li> <li>● After use, please clean and place in a dry place.</li> </ul>
 Unplug the power plug	<ul style="list-style-type: none"> <li>● When unplugging the power plug from the socket, do not pull the power cord directly, but hold the power plug and pull it out. → Otherwise, electric shock or short circuit may be caused, which may cause fire.</li> <li>● Be sure to unplug the power plug from the socket before cleaning and maintenance. Also, do not plug or unplug the power plug with wet hands</li> <li>● When not in use, be sure to unplug the power plug from the socket. → Dust or moisture may cause aging of wire insulation, which may cause electric shock, electric leakage or fire.</li> </ul>

## Other points for attention

Warning!	
	
 prohibit	<p>Do not stand on the massage chair. Do not give strong impact to the massage chair from above, such as sitting outside the chair seat. → Massage chair tilting may cause accidents, injuries or failures.</p>
 Do not disassemble	<ul style="list-style-type: none"> <li>● If the massage chair breaks down, please contact the sales shop or dealer and do not repair it by yourself.</li> <li>● If the power cord is damaged, please entrust the dealer who bought the product to replace it to avoid danger.</li> <li>● Do not modify the massage chair. → Otherwise, fire, electric shock or injury may be caused.</li> </ul>
Be careful!	
 force	<p>If you transfer or lend the massage chair to others, please be sure to hand over this manual together with the massage chair to the other party</p>

## Be careful!


When discarding this product and its accessories, please consult the nearest municipal window for the correct method of discarding.

## Name and function of each part

## Operating Instructions

※ The illustration is a schematic diagram. Different models have different shapes.

### Main of massage chair

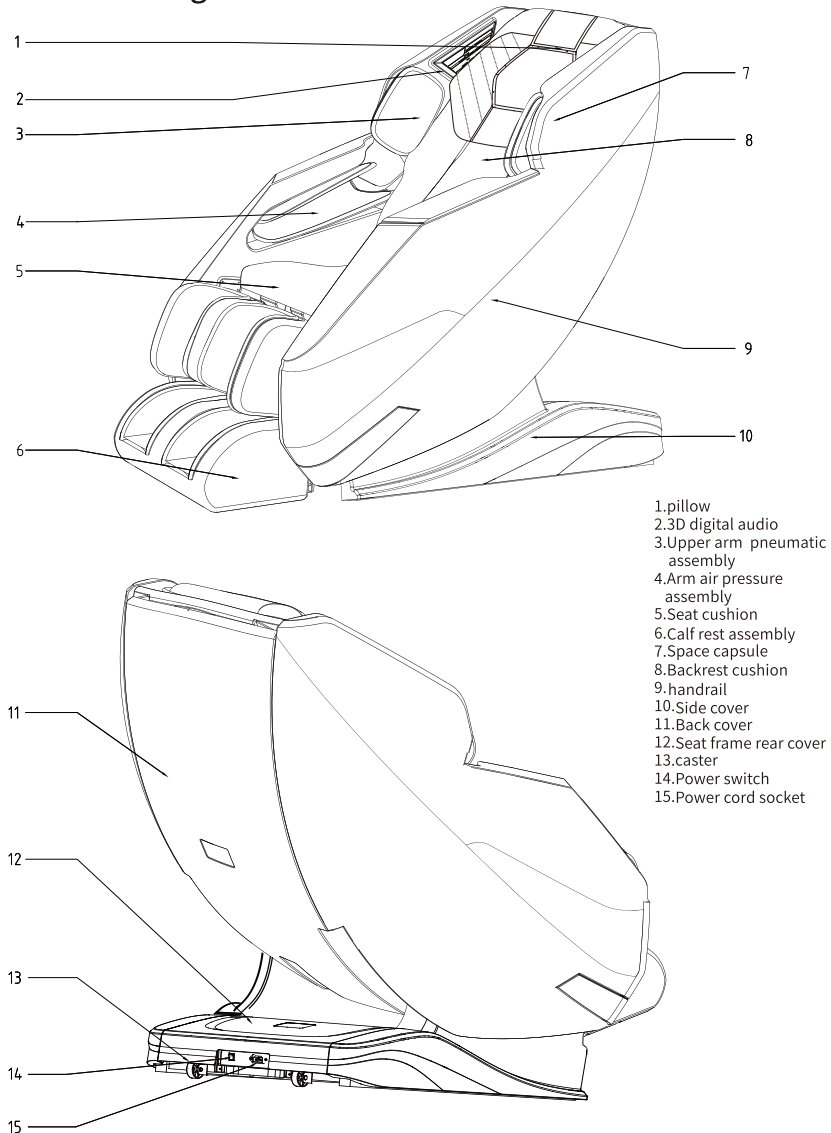

### Function

- This product is designed with a group of Intelligent massage manipulator that can move up and down, and calculate statically.
- It is equipped with automatic position detection and can adjust the height of the shoulder position according to which the human body position can be calculated to make the pressing more humanized and scientific.
- It is equipped with an 20 -motion mode, quick massage,Whole body relaxation, Full body stretch,Neck & Shoulder,Waist care,Spine care,Health recharge,Fatigue recover, Deep relief,Easy sleep,buttock care,Airbag massage,Swing mode,Elderly vitality,Cybraian mode,Lady mode,Worker mode,Dream mode,Japanese massage,Shiatsu massage.
- 3 Memory function.
- The manual mode function:Optional manual functions include massage position, massage technique, manipulation speed, manipulation width and sole roller.
- Air pressure massage:The air pressure mode is divided into full body air pressure, upper body air pressure, and lower body air pressure. The air pressure intensity is divided into three levels.
- It is equipped thermotherapy function: graphene is used as the far infrared heat source in the hyperthermia part to improve the blood circulation.
- The electric push cylinder and sliding technology are used under the backrest frame to keep zero space and save space to the greatest extent.
- The backrest support guide adopts the SL curve guide rail technology, which fully fits the back curve of the human body and enjoys the intimate massage from the head to the thigh.
- Wireless Bluetooth music function: 3D digital audio is placed inside the space to listen to music while massaging and release pressure.

# Operating Instructions

## Display screen of remote control




Note: The color, function and national standard typesetting sequence of massage function , please refer to the physical display

- |                                            |                                       |
|--------------------------------------------|---------------------------------------|
| 1. TFT color LCD                           | 11 Information                        |
| 2. Massage function status                 | 12 Manual display                     |
| 3. Time indication                         | 13. Back comfort display              |
| 4. Bluetooth display                       | 14. Massage position indication       |
| 5. Arm air pressure indicator              | 15 Zero gravity indication            |
| 6. Air pressure indicator of legs and feet | 16 Back heat                          |
| 7. Plantar roller                          | 17.Calfrest kneading speed adjustment |
| 8. Roller speed adjustment                 | 18 Massage position adjustment        |
| 9. Pressure intensity regulation           | 19. Width adjustment                  |
| 10. Speed regulation                       | 20. Shoulder position adjustment      |

# Operating Instructions

## Remote buttons

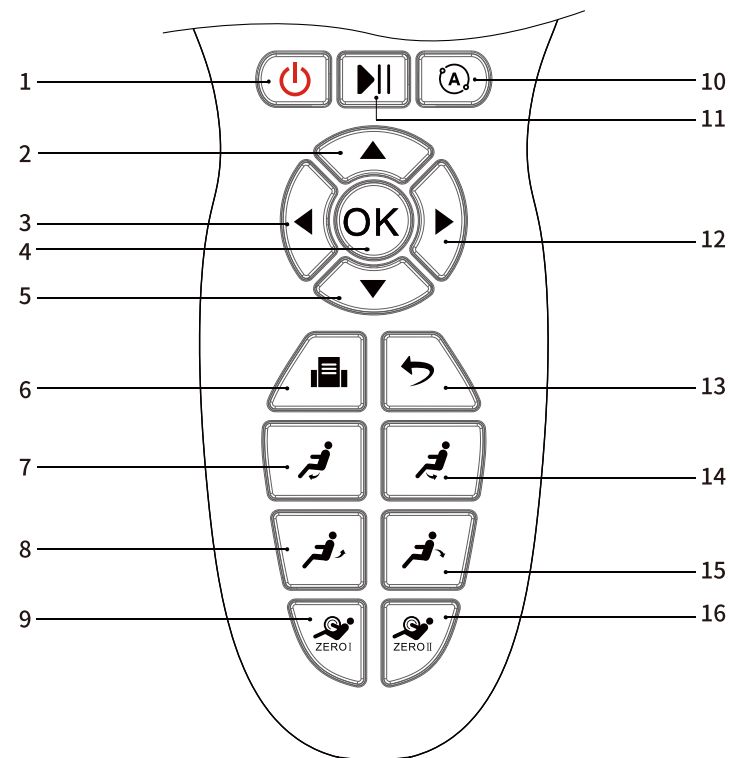


- |                                                                     |                                                                      |
|---------------------------------------------------------------------|----------------------------------------------------------------------|
| 1. On key: turn on or off the massage chair                         | 10. Auto:to enter automatic modes                                    |
| 2. Up key: navigation key up key                                    | 11. Pause key: pause the current function                            |
| 3. Left key: left key of navigation key                             | 12. Right click: navigation key right click                          |
| 4. K key: confirmation key                                          | 13. Return key: return to the previous level                         |
| 5. Down key: navigation key down key                                | 14. Lower key of lower leg frame: adjust lower leg frame             |
| 6. Menu key: enter the main menu interface                          | 15. Backrest down key: adjust the backrest down                      |
| 7. Lifting key of calf frame: adjust the lifting of calf frame      | 16. Zero gravity 2: adjust the massage chair angle to zero gravity 2 |
| 8. Backrest up key: adjust the backrest up                          |                                                                      |
| 9. Zero gravity 1: adjust the massage chair angle to zero gravity 1 |                                                                      |



# Operating Instructions

## I. Power on the whole machine

- 1 Schematic diagram of power on 2 Schematic diagram of connecting line of the whole machine 3 Schematic diagram of power switch of the whole machine

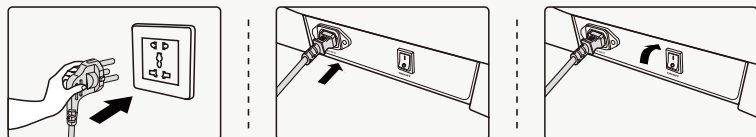

## II. Massage function control operation



### 1. Standby

- The massage chair is powered on and in standby mode. Power on interface: FUJILUX



### 2. Start massage (key on)



- Press the key to start massage.

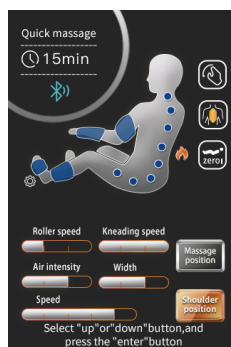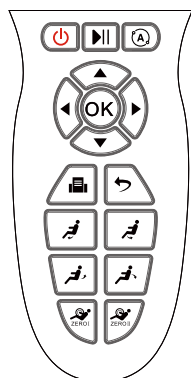

# Operating Instructions


### 1) Automatic adjustment

- After the massage is started, the massage chair will automatically enter the state. After the angle is adjusted, it will enter the shoulder position detection.

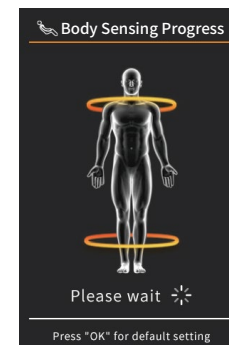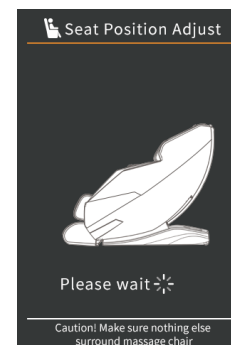

### 2) Shoulder position detection and shoulder position adjustment

After the manipulator detects the shoulder position, the buzzer will prompt "Didi". If there is a difference between the actual shoulder position and the shoulder position, you can press the "Up" or "Down" key to adjust it to the closed position. The shoulder position is adjustable for 11 gears. If there is no shoulder position adjustment within 10 seconds, it will automatically enter the massage function. If the "OK" key is pressed, it will directly enter the massage function. If the shoulder position is not detected, it will be massaged according to the default shoulder position height.

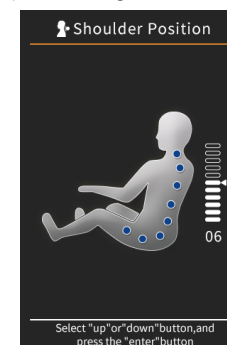


KEY	DESCRIBE
	Adjust the position of the massage ball upward to reach the full shoulder position.
	Adjust the position of the massage ball downward to make it reach the full shoulder position.

Trouble




# Operating Instructions

- After entering the massage, if the shoulder position height needs to be adjusted again, select the shoulder position adjustment interface through the navigation key "Up" or "Down" in the operation status interface, and adjust the shoulder position height through the "Up" or "Down" after entering the interface.

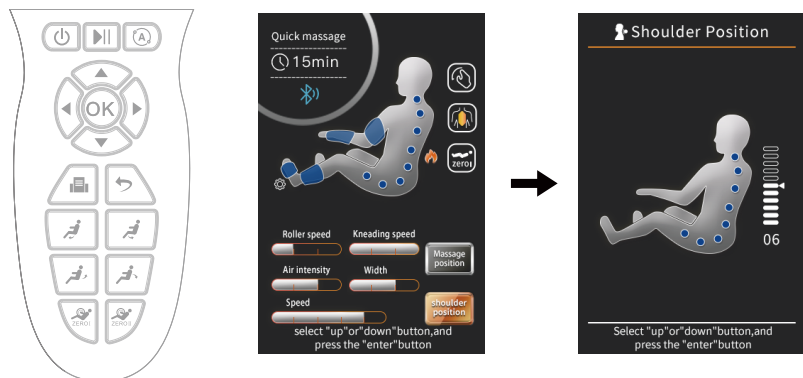

## 3) Massage position adjustment


After entering massage, the current technique is in the fixed point or local state of manual mode. If it is necessary to adjust the massage position, select the massage position adjustment interface through the navigation key "Up" or "Down" in the operation status information interface. Press and hold the "up key", the massage manipulator will walk upward, and if it is released, it will stop; Press and hold down the "down key", the massage manipulator will walk down, and it will stop if released.

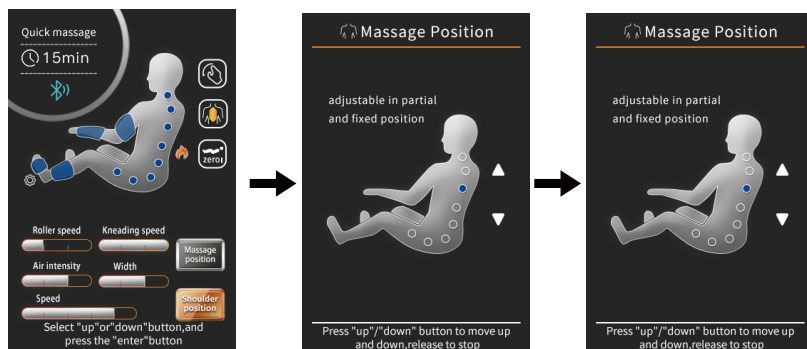

# Operating Instructions

## III. Pause key

When the massage function is running, press the "Pause" key to temporarily stop all massage functions, and press the "Pause" key again to resume the massage function before the pause; If the pause time exceeds 20 minutes, all massage functions will be turned off automatically.

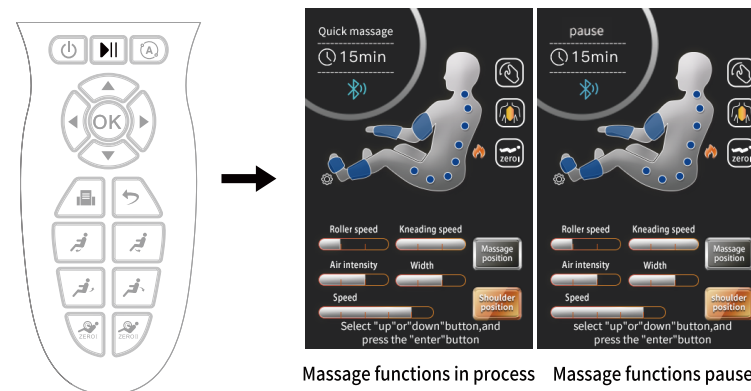

Massage functions in process    Massage functions pause

## IV. Menu key


- Press the "Menu key" to enter the main menu function selection interface, including automatic program, manual program, pneumatic massage, auxiliary function, memory, setting tool and other interfaces. "Navigation key" is selected, "Return key" is used to return to the upper menu, and "Confirm key" is used to confirm selection.



# Operating Instructions

## V. Automatic program(menu-automatic program)

• Press the "Menu key" to enter the menu interface, and select the icon of the automatic program. The automatic program includes eight modes: quick massage,Whole body relaxation, Full body stretch,Neck & shoulder,Waist care,Spine care,Health recharge,Fatigue recover, Deep relief,Easy sleep,buttock care,Airbag massage,Swing mode,Elderly vitality,Cybraian mode, Lady mode,Worker mode,Dream mode,Japanese massage,Shiatsu massage.

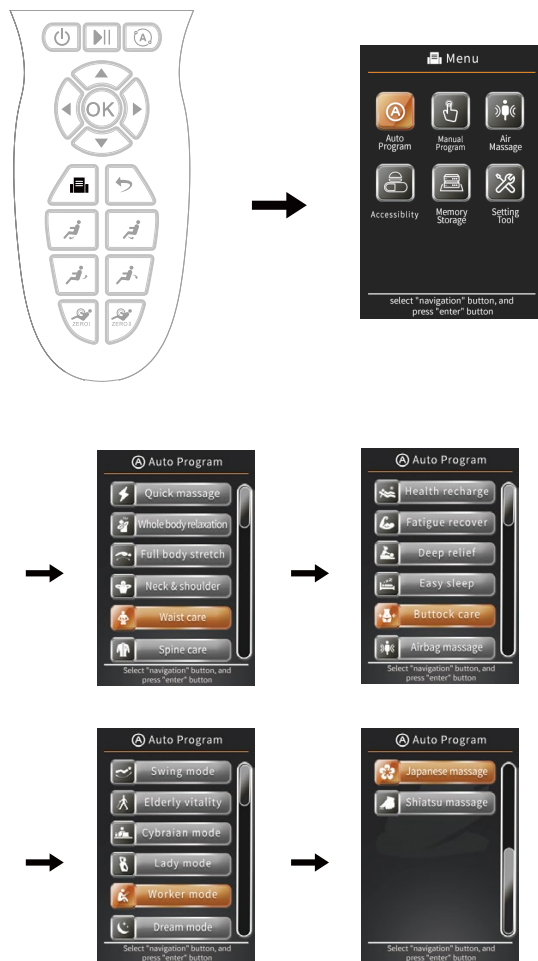

## VI. Manual program

• Press the "Menu key" to enter the menu interface. The player's moving mode icon: technique, position, speed, width, roller, etc.  
• Massage techniques include 360 ° kneading, smooth kneading, reverse kneading, upper half kneading, lower half kneading, tapping, kneading, finger pressing and tapping.  
• The massage positions include full back, upper back, lower back, local and fixed points.  
• There are 5 kinds of massage speed, and the speed can be selected only after the player moves.

There are 3 kinds of massage ball width options. In the manual mode, the width of the massage ball can be adjusted only when the massage technique is in the state of slapping, knocking and finger pressing.




Note: "Navigation key" is selected, "Return key" is used to return to the previous menu, and "Confirm key" is used to confirm the selection. Operations in other interfaces are similar.

# Operating Instructions

## VII. Air pressure massage

- Press the "Menu key" to enter the menu interface and select air pressure massage. The massage parts are divided into whole body, upper body and lower body, and the intensity is divided into three levels.
- The massage parts include full back, upper back, lower back, local and fixed points.

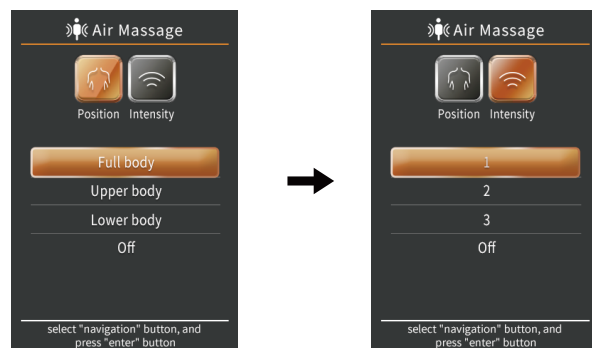

## VIII. Auxiliary functions


- Built in thermotherapy: use graphene heating to improve blood circulation.
- Foot roller: Foot roller massage, with 3 speeds, can scratch and massage the foot, promote blood circulation and accelerate metabolism.
- Calcrest kneading: calcrest kneading massage with 3 speed options. Massage the calves in all directions:



# Operating Instructions

## IX. Memory storage

- Press the "Menu key" to enter the menu interface, and select the memory function, including M1, M2, M3 and other three memory functions.
- Save: save the massage mode, massage technique, air pressure mode and intensity, the angle of the calf frame and backrest frame and other massage functions under the current operating state.
- Use: Select the previously stored massage mode, massage technique, air pressure mode and intensity, calf frame, backrest frame and other massage functions.

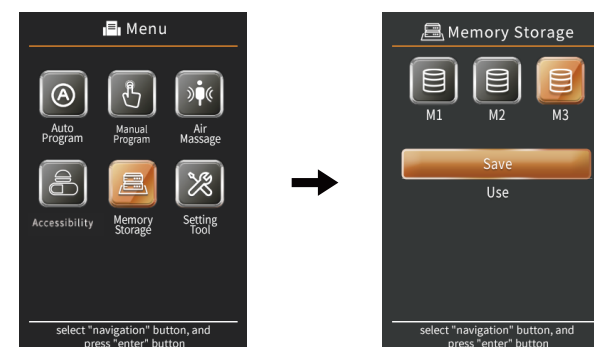

## X. Setup Tools


- Press the "Menu" key to enter the main menu interface, and select the icon of the setting tool, including time and language.
- The time setting is divided into three levels: 10 minutes, 20 minutes and 30 minutes.
- Language selection: multiple language settings.
- Bluetooth: turn on cellphone Bluetooth and match massage chair, music can be played once matching is successful.



## Before use

### Product installation


#### Unpack the packing box

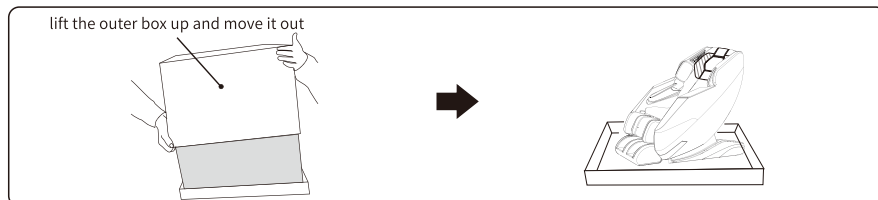

1. Disassemble the packing tape of the outer box of the chair body, lift the outer box up and move it out
2. Take out the accessories of the massage chair and put them in place

#### Moving Method

- Raise the backrest frame to the highest position
- When moving, lift the massage chair at the front end, press the massage chair down at the rear end, press the backrest, use the rear castors to move, and slowly lay the massage chair flat after moving to the required position

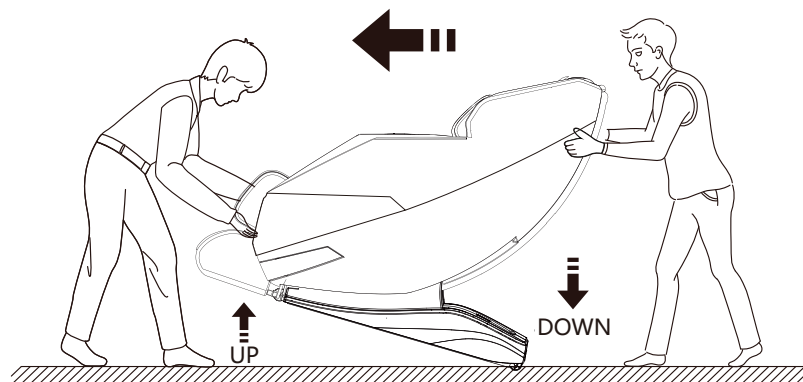


Note: When moving the massage chair, be sure to turn off the power switch first, and then unplug the power cord and the manual controller cable

## Before use

- The product is packed with the seat back completely flat  
When the room temperature is low, if you want to increase the indoor temperature and use the massage chair, please increase the temperature step by step. Do not increase the temperature too quickly. When moving the machine from a low temperature storage to a warm room, please wait about 1 hour before using it. → Otherwise, there may be condensation at the running part, and the performance cannot be fully developed


### About Places to Put the chair

Please ensure that there is more than 15-20cm space behind the massage chair and more than 40cm space in front of the massage chair to prevent the chair back from being tilted to the wall or other objects

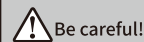

- Do not use it in the bathroom or sauna or other places with heavy humidity
- Do not press the power cord against the base of the massage chair
- Do not place the machine in direct sunlight or near heating appliances and other high temperature places
- Do not let cigarettes, ashtrays and other open flame objects near the massage chair
- Do not use it in the place directly facing the air conditioner

- In order to prevent damage to the floor and reduce running noise, please provide carpet and other materials under the massage chair (please do not provide electric blanket)
- As the massage chair is heavy, please be careful not to damage the floor when placing it on the ground
- Please place the massage chair 1m away from the TV, radio and other audio-visual electronic equipment. Otherwise, noise may occur.
- If a person with a weight of 100kg sits on a massage chair, the weight is about 3.9kg per 1cm

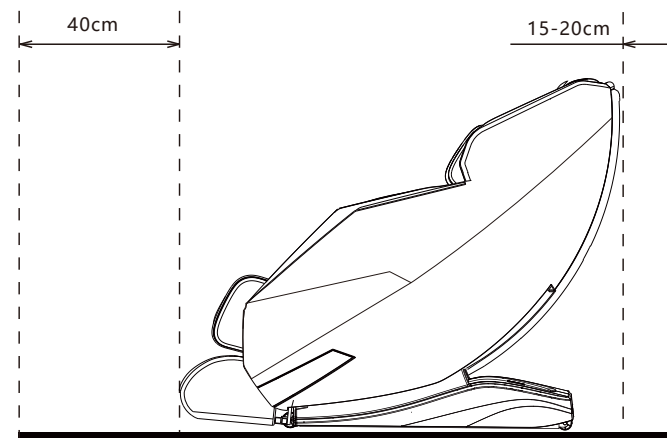


### Confirm the items in the packing box

Please confirm that the main body, power cord, instructions and other accessories are installed in the packaging box.

## Before use

### Confirm fabric

Open the packing box and confirm whether the cloth of the massage chair is damaged. In addition, please check other parts to ensure that the fabric is not damaged.

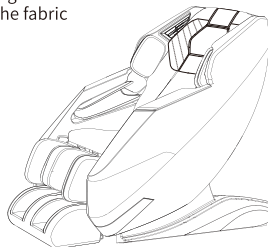


#### Warning!

Before massage, be sure to confirm whether the fabric at the massage part is cracked or damaged. Please check other parts to ensure that the fabric is not damaged. If you find any damage on the fabric, regardless of the size of the damage, please unplug the power plug and contact the dealer who purchased the product. Please never remove the fabric of the massage area or use it when the fabric is damaged, otherwise there will be serious potential safety hazards.

### Method of connecting power

- ① Insert the connection port of the power cord into the access port.
- ② Insert the power plug into the socket.
- ③ Turn on the main power switch behind the massage chair.

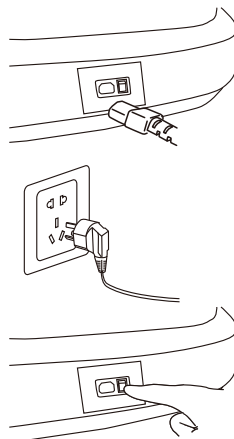

#### Warning!

- Do not use transformers  
→ Failure or electric shock may occur.
- Do not use it outside the country where you purchased it.  
→ Failure or electric shock may occur.



#### Be careful!

- Before inserting the power plug into the socket, make sure that the main power switch is in "O" state.
- Please insert the power plug fully.
- Before use, please follow this instruction manual, turn on all switches in turn, and confirm the normal operation of the machine.

## Trouble shooting

### Operating sound

The following operating sounds will be made during operation.

With the sound of hammering	Glug glug
Rubbing sound between massage wheel and fabric during kneading operation	Whoosh Squeak
The sound of massage wheel lifting, kneading and finger pressing	Buzz
The sound of air pressure massage	Clunk clunk
The sound of the charging pump	Bang Bang
Airbag exhaust sound	whoosh
The sound of the chair back tilting up and down	Buzz

※ The above sounds will not affect the performance

## Safety check

### When checking the product under long-term use

#### Whether the following situations occur

- \* Sometimes after pressing the switch, it still cannot operate.
- \* The power cord is cracked or damaged.
- \* The power cord and power plug are abnormally hot.
- \* There is a smell of burning or abnormal sound and vibration during operation.
- \* Sometimes the timer cannot be turned off automatically.
- \* Other abnormalities or faults occur.

#### Tips

If the situation described on the left side occurs, please stop using, unplug the power plug from the socket and send it to the dealer who purchased the product (recorded on the back cover) for inspection to avoid faults or accidents.

#### Love check



### For your safe use

The fabric of the massage part is consumables with a service life of 3 years. (Use conditions: 60 minutes per day.) For safety, it is recommended to contact the dealer who purchased this product before the service life is exceeded after purchase to check. If the service conditions are inconsistent with the above, the service life will also change. For safety, it is recommended to conduct a comprehensive inspection of the massage chair after 5 years of purchase. Please consult with the dealer when purchasing this product for the cost of inspection and maintenance.

### Q Can the massage chair be placed on the electric carpet?

**A** It may cause fire, so please do not put it on the electric blanket.  
As the massage chair is heavy, it may damage the heating element of the electric blanket, which may cause a fire.

### Q Can it be placed on wood floor and floor heating floor?

**A** Can be placed.  
However, the floor may be damaged, so it is recommended to place the massage chair on the carpet. (The same goes for tatami)

### Q How to enhance the massage intensity?

**A** Please try the following operation.  
● Fully sit down and place the weight on the back of the chair.  
● Adjust the massage force by pressing the force button of the controller.  
Through the above methods, the massage intensity can be increased.

### Q How to reduce the massage intensity?

**A** Please try the following operation.  
● Adjust the massage force by pressing the force button of the controller.

### Q What should I do if I spill water accidentally?

**A** Electric shock may be caused. Please stop using immediately and do the following.  
● Turn off the power switch behind the massage chair  
● Pull the power plug out of the socket

After the above operations, please contact the sales shop or dealer (recorded on the back cover) when purchasing this product

### Q Will the back rest recline automatically when massage finishes?

**A** It will not reset automatically when massage finishes if power is turned off  
However, when the power is turned off or the timer preventing overuse is turned on, the back rest will reset automatically


### Q Can people whose height is less than 150cm or more than 180cm also use it?

**A** Can be used.

### Q Can be use too heavy?

**A** It is recommended for people weighing less than 100kg

## Cleaning and maintenance methods

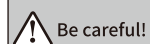


**Be careful!**

Be sure to unplug the power plug from the socket before cleaning and maintenance. In addition, do not plug or unplug the power plug with wet hands

## [Cleaning and maintenance of sewing products]

When stained with dirt, immerse the soft cleaning cloth in warm water with a small amount of neutral detergent, screw it completely and then wipe it.

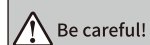

**Be careful!**

Discoloration may occur after long contact with plastic and indoor wallpaper.

## [Cleaning and maintenance of plastic parts]

Immerse the soft cleaning cloth in warm water containing a small amount of neutral detergent, and completely screw it dry before wiping. To avoid the residue of lotion, please use a dry cloth to wipe it clean again.

● Do not use alcohol, other solvents and polishing powder for cleaning and maintenance, otherwise it may cause scratches, discoloration and cracking.

Applicable goods	Not applicable goods
<p>water warm water detergent</p>	<p>Diluent Alcohol Other solvents Polishing powder</p>


## [Cleaning and maintenance of wooden panel, wooden pole and remote control]

Please use a dry cloth to wipe away the dirt on the wooden panel, wooden pole and remote control.

● Do not use a wet towel to wipe. → Otherwise, failure may occur.

## Storage method

- After decontamination and dust removal, place it in a dry place.
- When not in use, please cover the dust cover to avoid dust.
- To prevent pets from biting the power cord, do not let pets near the machine during storage

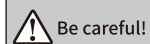

**Be careful!**

Do not place the machine in direct sunlight or near heating appliances and other high temperature places.



# After-sale service

## 1.warranty card


Please be sure to save the product warranty card provided by the sales shop or dealer at the time of purchase

Please read the warranty carefully and keep it properly. (Show it when repairing.) The warranty period is recorded in the warranty card. However, this product is only used in the country where you purchased it, so when it is used abroad, the warranty card is still invalid even within the warranty period. In addition, when it is used for other purposes than general household use, even within the warranty period, the relevant repair costs shall also be paid.

## 2.When the repair is entrusted

Before commissioning maintenance, if there is still any abnormality after inspection, please contact the sales shop where you purchased this product to commission the following matters, or pay attention to the wechat official account and contact information of the production company on the product warranty card

- 1) Name, address, telephone number
- 2) Product name and model
- 3) For the production number, please refer to the company label affixed on the chair foot cover.
- 4) Purchase date
- 5) Fault status (please inform in detail as much as possible.)

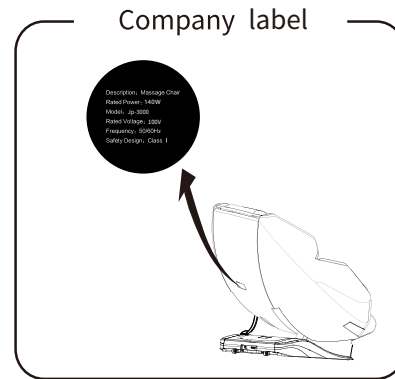

- Within the warranty period  
Please show the warranty card during maintenance. Please read the warranty card carefully because it may be a paid repair even within the warranty period.
- Beyond the warranty period expires  
Please contact the sales shop when purchasing this product. If the product performance can be maintained through maintenance, paid maintenance can be carried out according to customer needs.
- When it is impossible to entrust the sales shop  
Please contact your dealer.
- Minimum retention period of performance parts for maintenance  
Our company will keep the parts for at least 6 years after the production of the performance parts for maintenance of the electric massager is completed.  
The so-called maintenance performance parts refer to the parts necessary to maintain the product performance.
- If you have any questions about after-sales service  
Please consult the sales shop or dealer who purchased this product.
- When the product is used in a country other than the country where it was purchased, it may be refused to repair.

# Specifications

model	Jp-3000
Use power	100V~50/60Hz
Rated power consumption	140W
Rated frequency	50-60HZ
Rated time	
Kneading speed	Automatic program control
Beating speed	Automatic program control
Up and down movement speed of massage wheel	5.0cm/s
Massage range	1010mm
Massage wheel output	10cm
Barometric pressure	About 37kpa
Heater	Below 45 °C
Optional program	20 kinds
Manual program	Multiple coordination
Program timer	
Tilt angle	128 °~156 ° relative to the horizontal plane No grade (electric tilt)
Size of massage chair	About 143x76x114mm (L x W x H)
Weight	About 80kg
Enclosure	Power cord, manual and other accessories
packing	1 package/1 set